	B2515	B2530
1	Flat point screw M6 V06312	

### Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screw	Torques (Nm)
V06312	8

- Assemble the screws using normal (Ref. : Z31-243) locking compound
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

### Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- Use a damp cloth to clean the product

### Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

### EC markings

Group G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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**B2530**

### Designation

Tubular adapter with light alloy tube Ø 22 mm

- B2515 : Light alloy      Weight : 90 g      Length: 160 mm
- B2530 : Light alloy      Weight : 140 g      Length : 310 mm

### Purpose

Structural element used to make endoskeletal prostheses of the lower limbs

- B2515 et B2530 : Weight limit : 50 kg

### Adjustments

Tilt amplitude : +/- 9° compared to the reference axis.

Set the desired tilt using the 4 screws, in accordance with the limits set.

Lock the assembly using (Ref. : Z31-243) locking compound on the screws.

The tube can be cut again in length according to your needs.

Do not grip the tube in a vise !

Use tube cutter (Ref. : ZOC103) by applying it in comparison with the tube at right angles. Deburred carefully inside and outside of the tube with the reamer (Ref. : ZOC104).

Insert the tube to the stop of the adapter.