## Setting up

- Fix the adapter on the foot, draw the surround of the foot's base and cut along the line (schema 1)
- Break the angles like schema 2
- For the sticking, sand down the junction plate's contact area, use the stick ref. Z3602 and follow the instructions for use

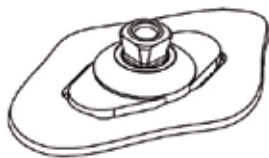
Schéma 1

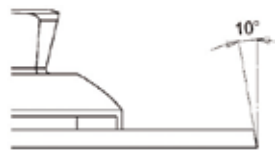
Schéma 2

## Restriction on use / Precautions for use

- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved orthoprosthesis depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved orthoprosthesis depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

## Cleaning / Storage

- Store the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthesis depending on the patient concerned

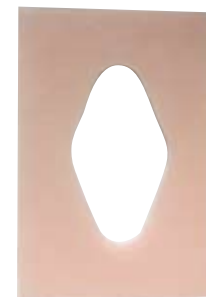
## Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.


*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

## EC markings

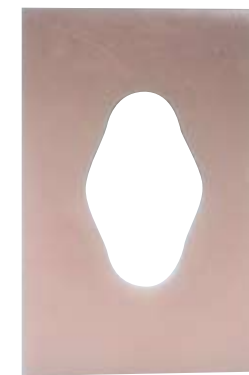Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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