Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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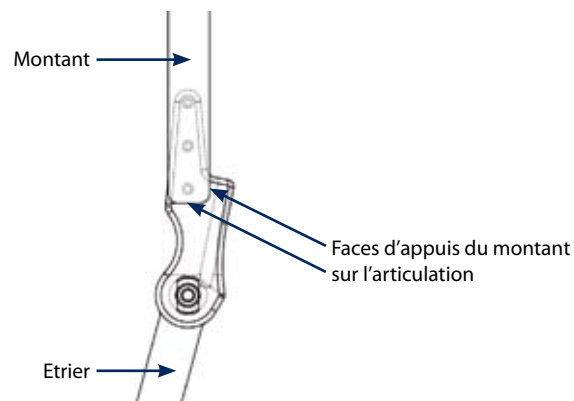
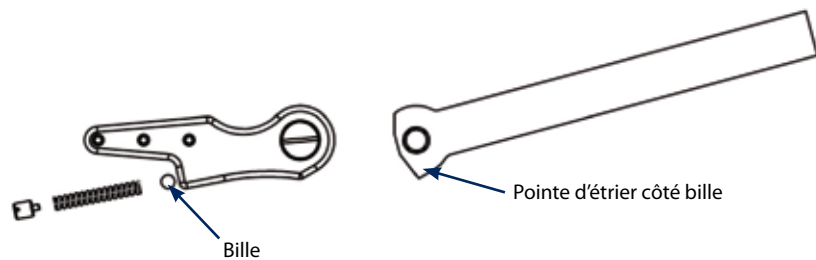
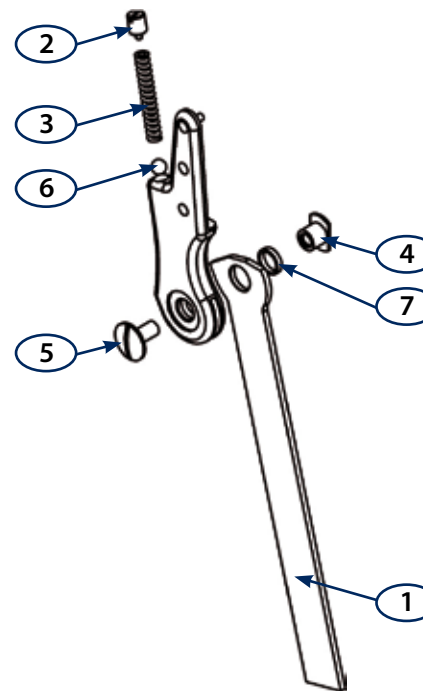
Restriction on use / Precautions for use

- Do not heat the brace joints to temperatures above 300 °C, as this can damage them
- Tip : line the brace joints up as well as possible to reduce wear on the contact surfaces
- Lubricate the moving parts with Molyckote Z10 grease
- The half-stirrups are welded or brazed after the brace joint has been removed
- Do not bend the aluminium brace joint
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

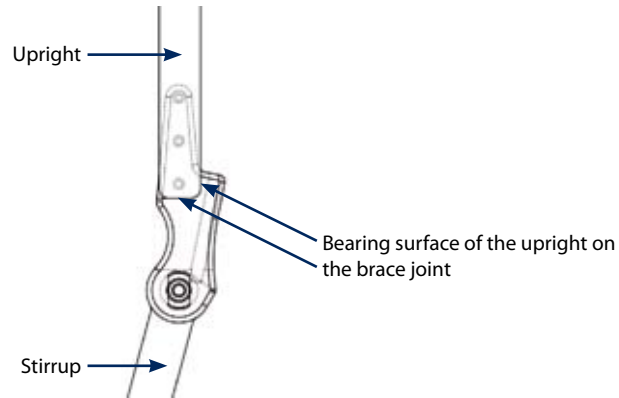
Fitting

Rivets are used to assemble uprights on the brace joints.

- ↪ Shape the upright so that it fits flush with the brace joint (see diagram)
- ↪ Drill through the upright, lining the holes up with those on the ankle brace joint

For carbon uprights, use JAD rivets.

- ↪ Cold shape each half-stirrup after removing it from the brace joint.


Assembly


To optimize your assembly work, we recommend using these parts in conjunction with other parts of the same class in the JANTON range.

Adjustment

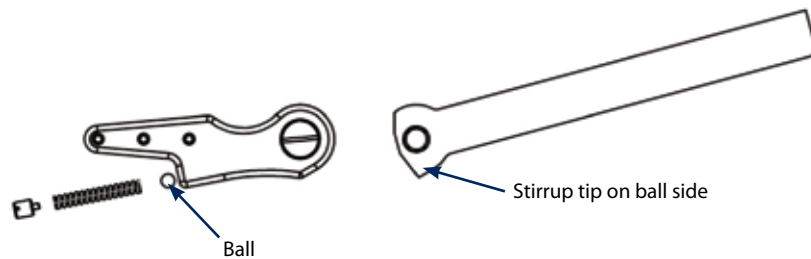- ↪ Turn screw (2) to adjust spring compression.
- ↪ Position the stirrup head as shown in the illustration below :

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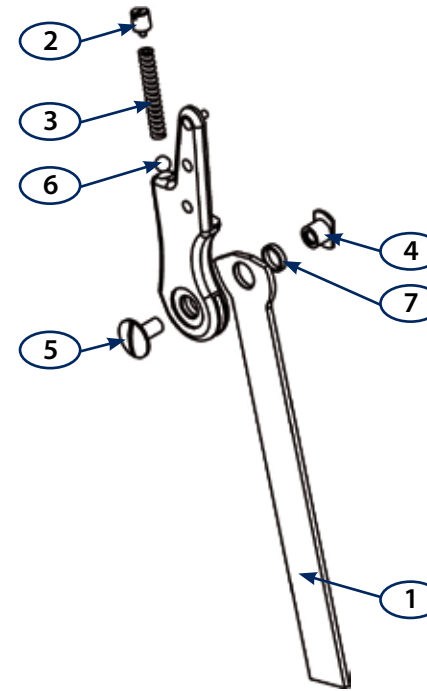1	Demi-étrier	L68
2	Vis de ressort	L1903
3	Ressort	L1904
4	Culot	V0012
5	Vis de culot	V0013
6	Bille	V0024
7	Bague en bronze	L1905

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Designation

Pair of Klenzac type aluminium ankle brace joints with springs and straight stainless Klenzac steel half-stirrups.

Pair supplied with 6 rivets (V18312).

Purpose

Ankle brace joint for use in lower limb orthosis.

Compatibility

Uprights 18 x 4 mm