Fitting


To obtain correct simultaneous locking, link metal rod after alignment (use link socket (6) per soldering).
Insert the upright inside the joint so that it lays flat. Rivets countersink mustn't weaken the joint.
To optimize your assembly work, we recommend using these parts in conjunction with other parts of the same class in the JANTON range.

Adjustments

When making the orthosis, it is highly advisable to optimize joint alignment to reduce wear of the contact surfaces.

NOTE : Control joint parallelism in order to reduce wear of surfaces in contact.

Spares parts

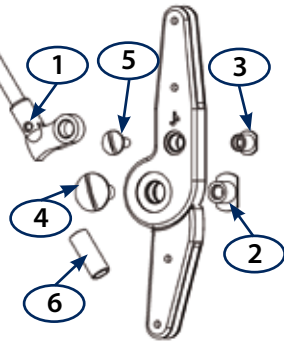		M08D20	M08D18	M09D20	M09D18
1	Lock	M080820	M080818	M090820	M090818
2	Joint cap	V0012	V0015	V0012	V0015
3	Lock cap	V0028	V0028	V0028	V0028
4	Joint screw	V0013	V0016	V0013	V0016
5	Lock screw	V0027	V0027	V0027	V0027
6	Link socket	M1006R	M1106R	M1006	M1106R
	Rivets	V18313	V18312	V18313	V18312

Restriction on use / Precautions for use

- Do not heat the joints
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved orthoprosthesis depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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