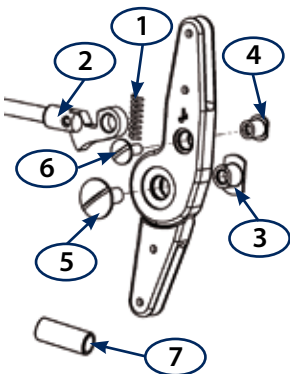
Fitting

To obtain correct simultaneous locking, link metal rod after alignment (use link socket (7) per soldering).
Insert the upright inside the joint so that it lays flat. Rivets countersink mustn't weaken the joint.
To optimize your assembly work, we recommend using these parts in conjunction with other parts of the same class in the JANTON range.

Adjustments

When making the orthosis, it is highly advisable to optimize joint alignment to reduce wear of the contact surfaces.
NOTE: Control joint parallelism in order to reduce wear of surfaces in contact.

Spares parts

		M10	M11	M12	M10SR	M11SR	M12SR
1	Spring	M1007	M1107	M1207			
2	Hoffa lock	M1008	M1108	M1208	M080820	M080818	M1208SR
3	Joint cap	V0012	V0015	V0015	V0012	V0015	V0015
4	Lock cap	V0028	V0028		V0028	V0028	
5	Joint screw	V0013	V0016	V0016	V0013	V0016	V0016
6	Lock screw	V0027	V0027	V0017	V0027	V0027	V0017
7	Link socket	M1006R	M1106R	M1106	M1006R	M1106R	M1106R
	Rivets	V18313	V18312	V18310	V18313	V18312	V38310

Restriction on use / Precautions for use

- Do not heat the joints
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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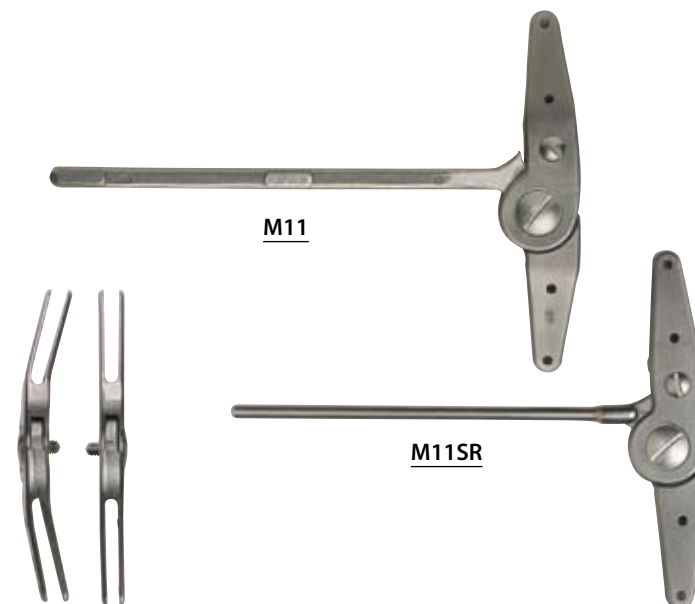
Joint with or without return spring (SR)

- ⤴ M10SR : Strong set Stainless steel Weight : 320 g
- ⤴ M11SR : Light set Stainless steel Weight : 240 g
- ⤴ M12SR : Child set Stainless steel Weight : 181 g

Each pair is composed by an internal arched joint and an external straight one.

Purpose

Knee joint for use in lower limb orthosis.

Compatibility

- ⤴ M10 and M10SR : Rod cross-section 20 x 5 mm
- ⤴ M11 and M11SR : Rod cross-section 18 x 4 mm
- ⤴ M12 and M12SR : Rod cross-section 16 x 4 mm