## Composition

2 metal fittings (internal and external).

## Spares parts

	M52	M53	M54
Articulation screw	V0108	V0107	V0107
Ball bearings	V406196	V405165	V405134

## Restriction on use / Precautions for use

- ⌚ The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ⌚ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ⌚ All assembly and adjustment work must be carried out by qualified personnel
- ⌚ Heat the part with a blowtorch before modifying its shape (any cold shaping will nullify the guarantee)

## Cleaning / Storage

- ⌚ Stock the product in a dry place, in its original packing (on which the batch number is marked)
- ⌚ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

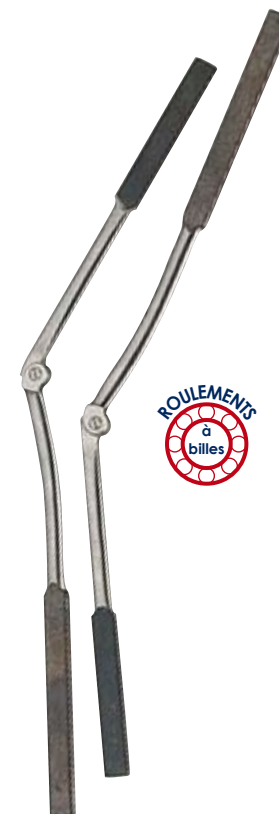
## Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

## EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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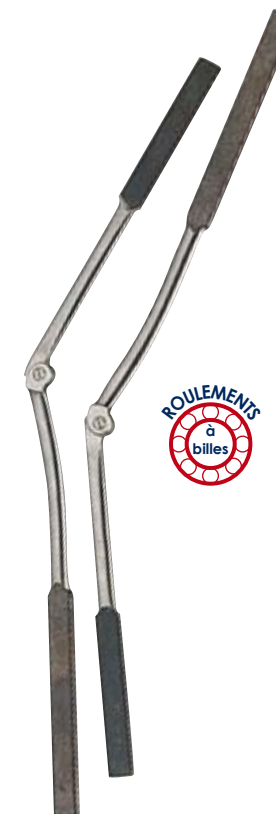
## Designation

Fitting in semi-hardened steel for walking apparatus, covered head with no lock, upper and lower uprights with reserve.

- M52: Strong series for an upright 22 x 5 mm section  
Reserves : 30 x 5,5 mm      Ball bearings : 6 mm
- M53: Light series for an upright 20 x 4 mm section  
Reserves : 28 x 4,5 mm      Ball bearings : 5 mm
- M54: Child series for an upright 18 x 4,5 mm section  
Ball bearings : 4 mm

## Purpose

Knee articulation used for the manufacture of orthosis for a lower limb.