Composition

1 internal fitting + 1 external fitting

Spares parts

	M96	M97	M98
Screws	V03406 V0156T	V03406 V0157T	V03405 V0157T
Bearings	V406196	V405165	V405134
Lock	M8809	M6509	M2707
Bal bearings	V0024	V0024	V0024
Spring	M2501	YRC0548	YRC0548
Rivets	V17315	V17312	V17312

Restriction on use / Precautions for use

- ⤿ The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ⤿ At the time of shaping the thigh and leg amounts, be careful not to deform the mechanism of the joint
- ⤿ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ⤿ All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- ⤿ Stock the product in a dry place, in its original packing (on which the batch number is marked)
- ⤿ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned


Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

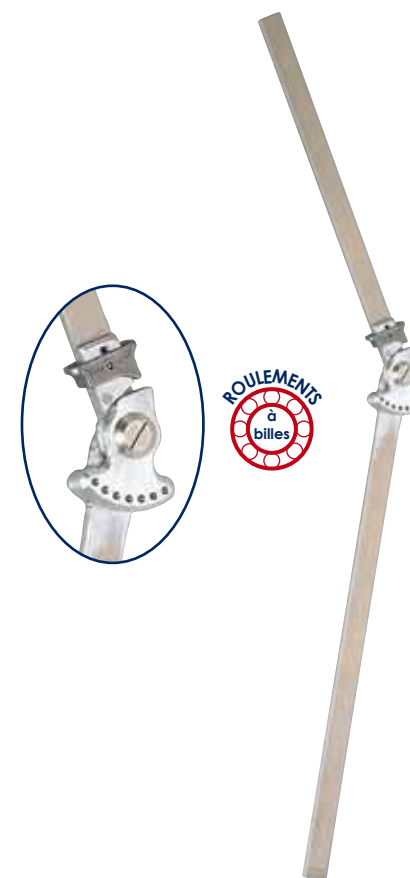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