
## Adjustments

Set the desired angulation using the 4 screws, in accordance with the limits set.

## Spares parts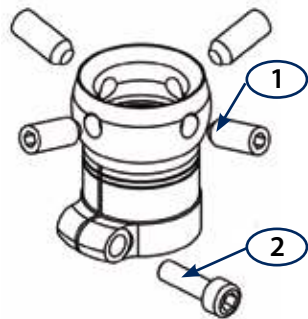

		B20STFX	B20STMX	B20TI
1	Flat point screw M8	V03I-0816	V03Z-0816	V03IAZ-0812
2	CHC screw M6	V02I-0620	V02Z-0620	V02IAZ-0516

## Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screw	Torques (Nm)
V03I-0816 and V03IAZ-0812	13
V02I-0620	12
V02IAZ-0516	10

Screw	Torques (Nm)
V03Z-0816	15
V02Z-0620	13

- Assemble the screws using normal (Ref. : Z31-243) locking compound
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

## Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

## Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

## EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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