Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screws	Torques (Nm)
V02I-0620 (7)	12
V03TI-0616 (4)	5
V02I-0825 (5)	15

- ⌋ Assemble the screws using normal (Ref. : Z31-243) locking compound
- ⌋ In the lower section, the tube must be inserted as far as possible in the bored hole
- ⌋ This C20 knee must not be fitted in cases of recurvatum, so it is not suitable for patients with hip dislocations
- ⌋ The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ⌋ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ⌋ All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- ⌋ Stock the product in a dry place, in its original packing (on which the batch number is marked)
- ⌋ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- ⌋ Use a damp cloth to clean the product

Guarantee

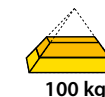
The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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


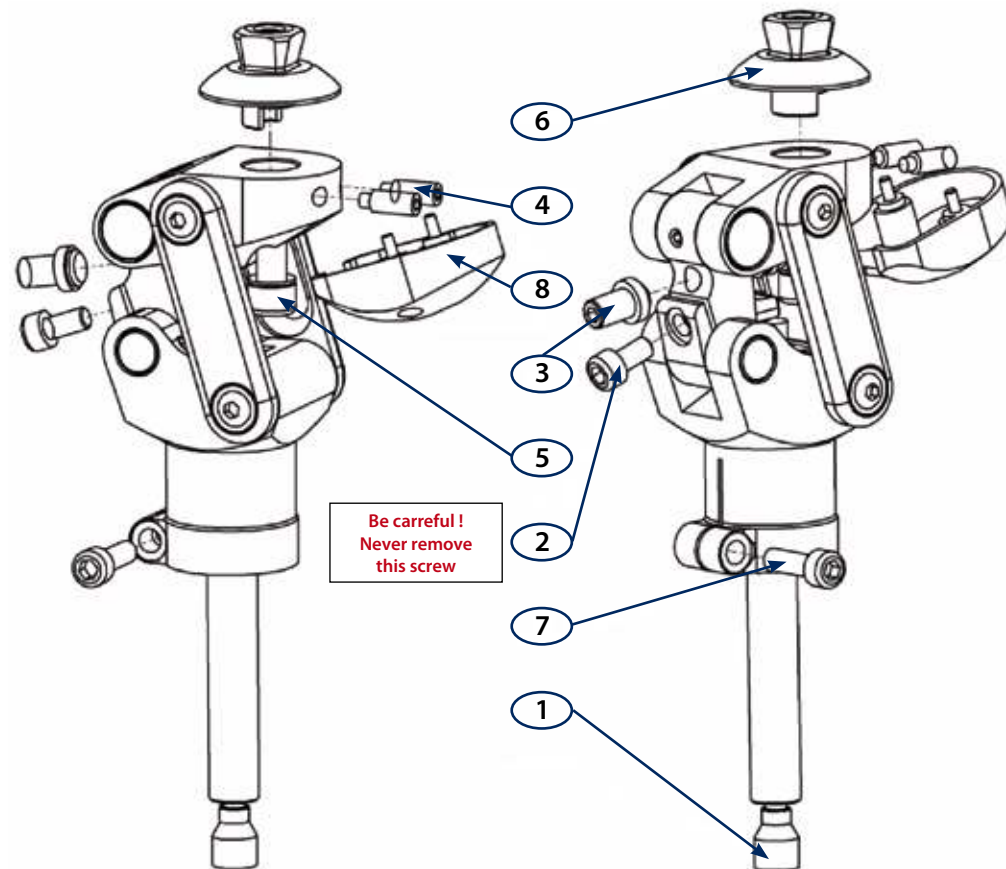
Spare parts

1	Spring compression adjustment screw	V03CI-1212
2	Extension braking adjustment screw	V02I-0612
3	Instantaneous rotation centre adjustment screw	C2017
4	Pyramid angulation adjustment screw	V03TI-0616
5	Pyramid tightening screw	V02I-0825
6	Pyramid	C302501
7	Tube locking screw	V02I-0620
8	Protective cover	C2030


Adjusting the centre of rotation

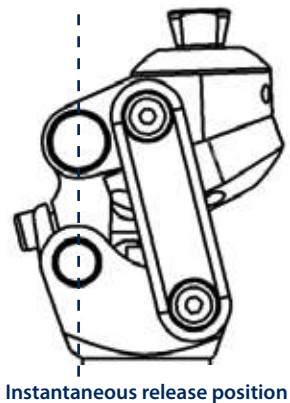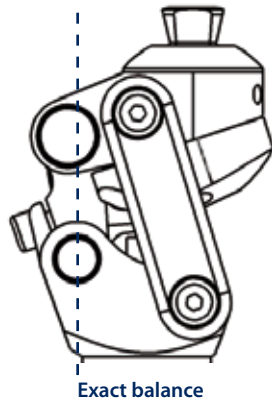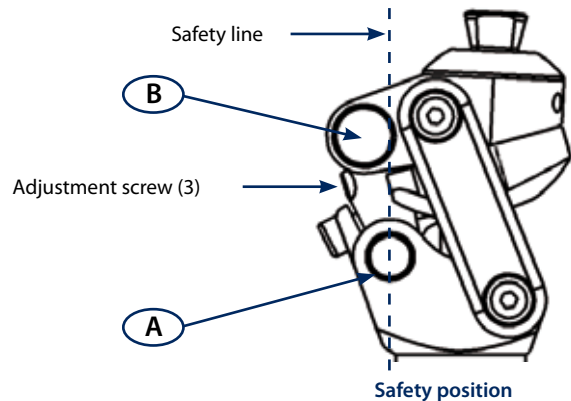- ⤴ The safety line is a vertical line going through the centre of axis A
- ⤴ The screw (3) can be adjusted to give the knee prior flexion, thus providing a faster triggering cycle
- ⤴ As the screw (3) is tightened, the centre of joint B moves closer to the safety line, and knee movement is triggered faster. When the centre of joint B has moved beyond the safety line, the knee is released instantaneously

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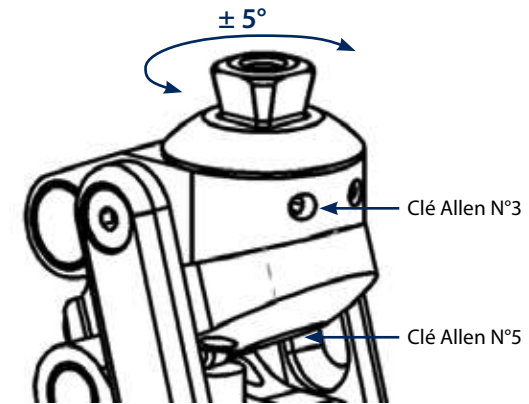
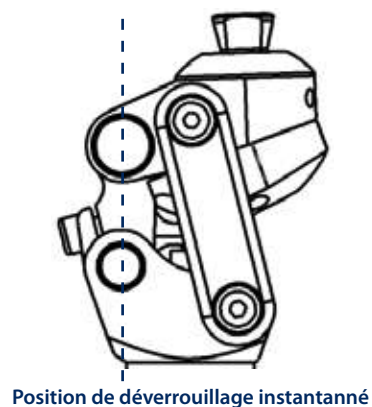
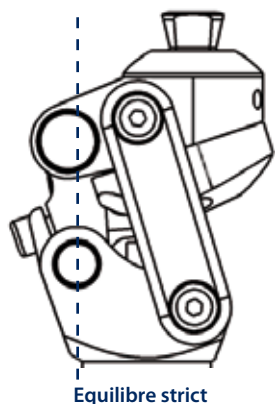
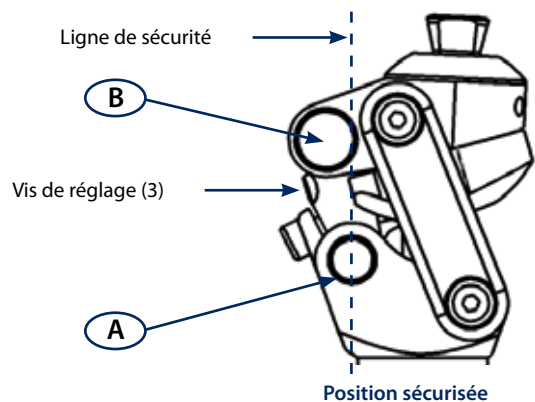
Adjustments

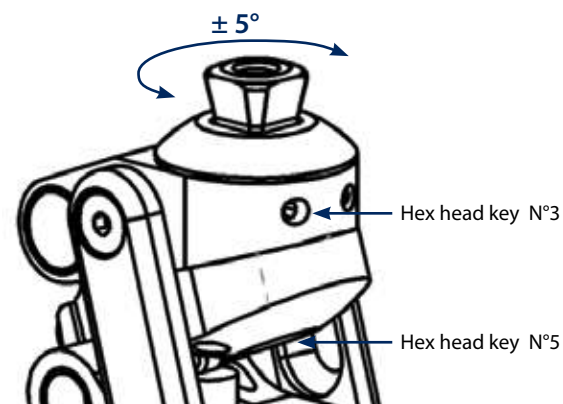Connector angulation adjustment : maximum angulation $\pm 5^\circ$

- Loosen the screw (5) on the connector
- Loosen the 2 pressure screws (4)
- Adjust the angulation of the connector (6)
- Tighten all the screws (apply (Ref. : Z31-243) locking compound)

Adjusting the extension return spring

- The speed of the extension return system can be adjusted by tightening or loosening movement on the screw (1) modifying the spring compression level

Adjusting the extension braking system

- The extension braking system can be adjusted to suit the patient receiving the prosthesis, via the tightening screw (2)
- Never remove the screw (2)**

Compatibility

- Upper section : connectors in the STARFLEX range
- Lower section : 30 mm diameter tube

Connectors in the STARFLEX range (upper section)

CA1
Male pyramid


CA3
Connector
for temporary fitting

CA7
Rotary adaptor with anchor
for cementing, large branches


Setting up

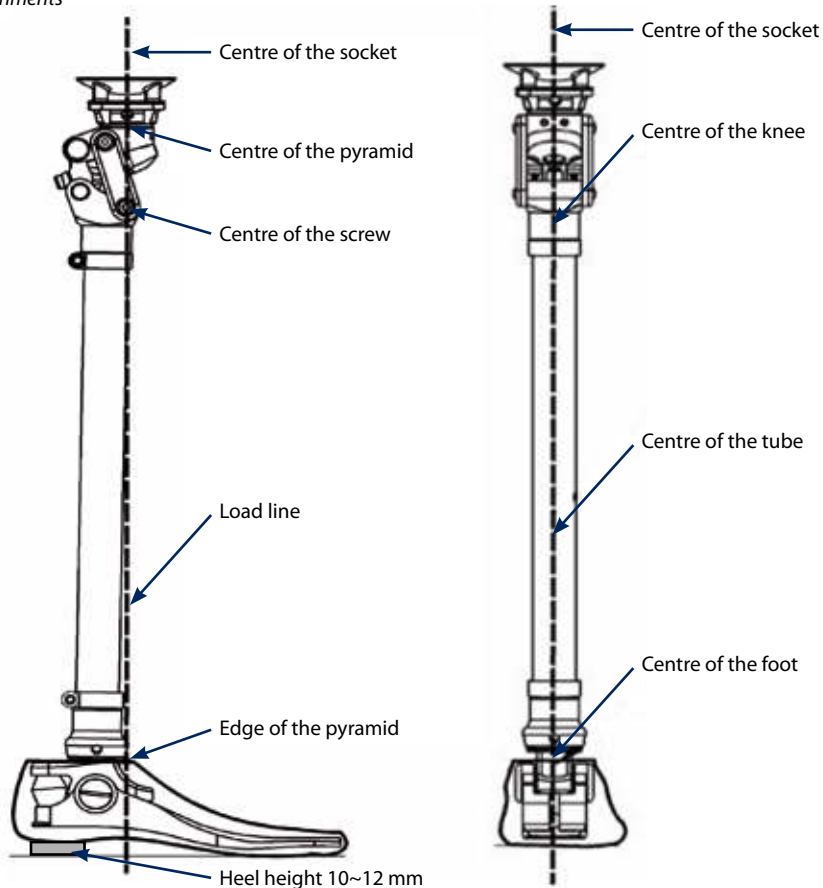Theoretical alignments

Important note !
A C20 knee must not be fitted in cases of recurvatum,
so it is not suitable for patients with hip dislocations.

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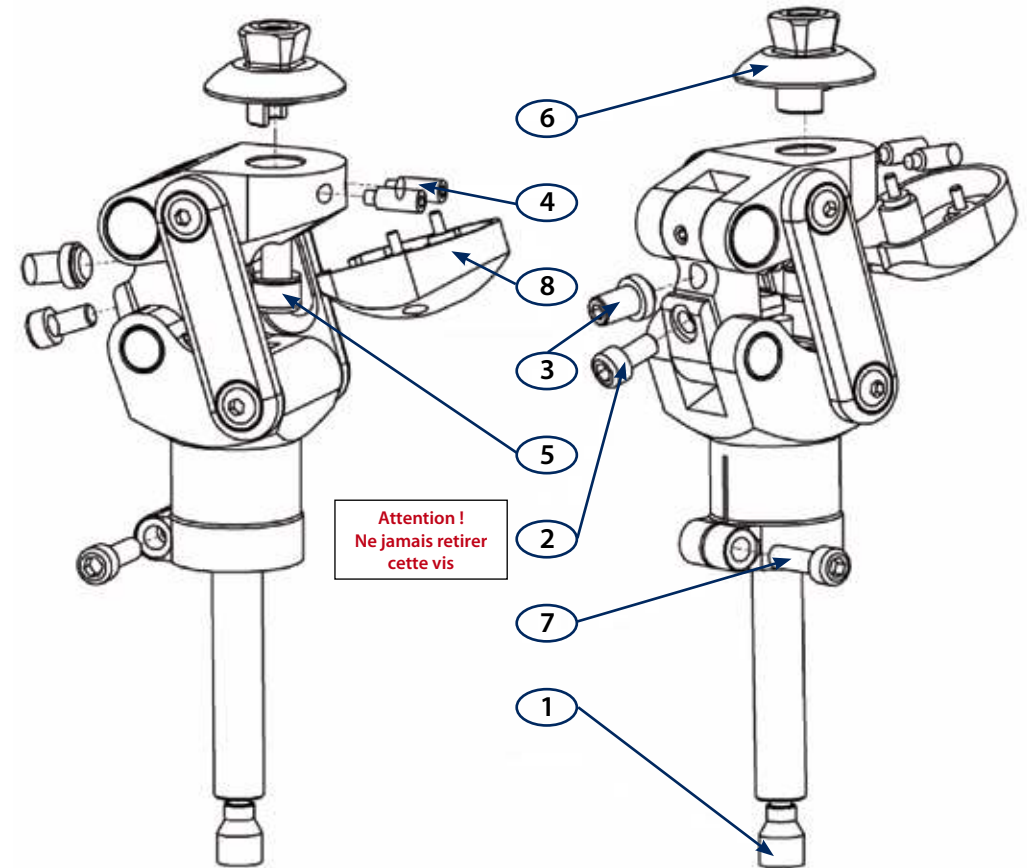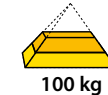
ST AIRFLEX

Désignation

Spring assisted polycentric knee joint

- ⌋ Weight : 485 g
- ⌋ Adjustment of extension return breaking
- ⌋ Joint fitted with ball bearings

Purpose

Knee used to make endoskeletal prostheses for lower limbs

- ⌋ Maximum patient weight : 100 kg
- ⌋ Patient with moderate activity. Limited walking area. Indoor/outdoor use